



LUVIN UNIVERSE

Document: Instruction for use

Drafted	Verified	Approved
GIUSEPPE DI MARTINO	FURONE MARIANNA, HUANG MAX	CHEN XIAOLI
Date: 19 June 2025	Date: 24 June 2025	Date: 24 June 2025

REVISION		
Revision	Reason for revision	Date
0	First issue	June 2025
1	Certificate revision	May 2026

Instructions for Use

Donning:

- Check the integrity of the packaging and gloves before use.
- Hands must be clean before wearing the gloves.
- Wear the gloves in the correct order to avoid the transmission of infections.
- Remove the gloves from the packaging. Once the first glove has been removed from the packaging, wear it carefully touching only the cuff area.
- Remove the second glove with the other hand and wear it carefully touching only the cuff area.
- Before use, ensure that the gloves fit properly in order to wear them comfortably.
- Once the gloves are worn, do not touch anything that could contaminate the gloves.
- Avoid tearing: pointed or sharp objects may puncture the gloves. Tearing may cause splashes of contaminants into the eyes, mouth, skin, or onto nearby persons.

Doffing:

- When removing the gloves, avoid contact between the outer surface of the gloves and the skin, as the surface may be contaminated with blood or other body fluids.
- Using the dominant hand, users start by grabbing the outside of the glove on the non-dominant hand on the palm side near the cuff.
- Pull the glove off the non-dominant hand and place it in the gloved hand, balling it up.
- Slip two fingers under the cuff of the other hand glove and carefully peel it off the hand without touching the wrist, turning the remaining glove inside out as it is removed and in turn encasing the first glove.
- The gloves can be disposed.

- Do not use in case of visible defects such as damaged or punctured nitrile.
- Use as needed and until the device deteriorates (tears, damage, etc.). In any case, do not use for more than 6/8 hours.

Additional Instructions

- This product is for single use only and must not be reused. Always discard and replace the product immediately after removal.
- The product cannot be washed, dried, or ironed.